

Global health  **safety**[®]
born to **care.**

 **Global Health and Safety Ltd.**

Unit K3, Pitfield, Kiln Farm,
Milton Keynes, MK11 3LW, UK

 Zhende Medical Co., Ltd.
Gaobu Town, 312035, Shaoxing, Zhejiang, CHINA
 Shanghai International Holding Corp. GmbH (Europe)
Eiffestraße 80, 20537, Hamburg, GERMANY

customers@globalhealthandsafety.co.uk
www.globalhealthandsafety.co.uk



Medical Device



Conformité
Européenne



Caution



Instructions for Use



EO sterilization



Single sterile
barrier system



Single use only



Do not resterilize



Keep Dry



Keep away
from sunlight



Do not use
if package is damaged



Catalogue number



Batch Code



Distributor



Manufacturer



Authorized representative
in the European
Community/European Union



Use by date



Date of
manufacture

Global health  **safety**[®]
born to **care.**

Flexy  **SUPERABSORBENT DRESSING**

Super Absorbent Dressing Sterile



Instructions for Use

GHSIFU08.V.01
11/2025

Product Description

Flexy superabsorbent is a sterile, multi-layered dressing designed for the management of wounds with moderate to high levels of exudate. It consists of a breathable, fluid-repellent nonwoven backing, a core with superabsorbent particles, and a nonwoven wound contact layer. The dressing effectively absorbs and retains wound exudate within the superabsorbent core, helping to maintain an optimal moist wound environment that promotes faster wound healing and closure. The core's high absorption capacity minimizes the risk of maceration by locking exudate away from the wound and peri-wound skin. Additionally, the dressing supports soft debridement by gently removing nonviable tissue during dressing changes. Free from adhesives and pharmacologically active substances, provides a secure, comfortable solution for wound care.

Intended use:

The Flexy Silicone Foam Dressing is adaptable to different phases of wound especially to all kinds of highly exudated wounds and all kinds of post-operation surgery wound and injuries, such as burn wound, ulcer, pressure ulcer, refractory chronic wounds.

Indications for Use

Flexy Super absorbent dressing is intended for use on both chronic and acute wounds with moderate to high levels of exudate such as:

- Leg ulcers, pressure ulcers, diabetic foot ulcers
- Post-operative wounds
- Superficial or partial thickness burns
- Traumatic wounds

Note: If unsure whether this product is suitable for a specific wound type, it is advised to seek the guidance of a qualified healthcare professional.

Contraindications

- Should not be used on patients with known allergy or hypersensitivity to any of the materials or components of the product.
- Should not be used on low exuding wounds, as well as on eyes, mucous membranes, or within wound cavities
- It should not be used on third-degree burns.
- Should not be used on dry or hard black necrotic wounds.

Warning and Precautions

- This product is disposable. Do not use if the inner package is damaged.
- This product is sterilized. Use immediately after opening the packaging, retain a single piece of the product for no longer than 7 days.
- This product shall not be reused.
- The products have a shelf-life of 3 years. Do not use beyond the expiration date.
- Change the dressing promptly when it swells and approaches the edges after absorbing exudate.
- Store the product in a cool, dry place, protected from light, and ensure adequate ventilation.

Side-effects

There are no known side effects associated with the use of this product.

Directions for Use

1. Preparation

- Wash hands and apply clean gloves.
- Cleanse the wound according to clinical protocols.
- Ensure the wound area is cleansed and the surrounding skin is clean and dry.
- Select an appropriate dressing size that is large enough to provide approximately 2 cm from the edge of the contact pad with healthy, dry peri-wound skin, ensuring full coverage.
- The Super Absorbent Dressing is not a haemostat and should not be used on bleeding wounds or arterial bleeds.

2. Application

- Carefully open the dressing pouch using Aseptic Non-Touch Technique (ANTT) to avoid contaminating the dressing. Do not touch the wound pad directly
- Apply the Super Absorbent Dressing directly to the wound, with the white side in contact with the wound and the blue side facing away.

- For large wounds, multiple dressings can be applied side-by-side.
- The dressing can be used either as a primary dressing in direct contact with the wound or as a secondary dressing over an absorbent primary dressing.
- Secure the dressing in place using medical tape, retention bandages, or dressings, ensuring adequate fixation while accounting for expansion during absorption.

3. Removal

Correct removal of the dressing is essential to protect the wound bed and preserve newly formed granulation tissue.

- Carefully remove the retention bandage or any secondary dressing has been used to secure the dressing.
- Start by gently lifting one corner of the dressing away from the wound bed, being careful not to pull it directly off the wound. Ensure that the dressing does not adhere to the wound bed.
- If needed, moistening the dressing with saline solution can help make removal easier and reduce discomfort. This can also help prevent injury to the newly formed tissue.
- As the dressing is removed, support the wound site to prevent any stress or strain on the newly exposed skin.
- Once removed, discard the soiled dressing in the appropriate clinical waste container, following local regulations.

Frequency of Change

- The use of super absorbent dressings should always be re-evaluated if wound exudate levels change.
- During the early stages of wound management, observe the wound daily for signs of infection or other complications (or as recommended by local clinical protocols).Infected wounds should be inspected and treated as per local clinical protocol.
- The frequency of dressing change is determined by the condition of the wound and local clinical protocols.
- Dressings can be left in place, undisturbed, for up to 7 days depending on wound conditions and saturation of wound dressing.

Storage and Shelf Life

- The dressing has a shelf life of 3 years and should not be used beyond the indicated expiry date.
- It should be stored lying flat in a cool, dry place, away from direct sunlight.

If any serious incident has occurred while using this device, please report to national authority and to the manufacturer.